VALMY INDUSTRIES SAS
Rue Benjamin Franklin,
42300 Mably - France
Tel. +33 (0)4 77 23 11 21 Fax +33 (0)4 77 72 54 57
info@valmy.eu - www.valmy.eu

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INSTRUCTIONS FOR USE
GEBRAUCHSANWEISUNG
INSTRUCCIONES DE USO
ISTRUZIONI PER L'USO

CE 0082
EN 149 : 2001+ A1 : 2009

Conforme au règlement EPI 2016/425
Conform to PPE regulation 2016/425
Entspricht der PSA-Verordnung 2016/425
Cumple con la normativa EPI 2016/425
Conforme al regolamento DPI 2016/425

Certification CE et contrôle de production annuel Module D par CE certification and annual production control Module D by CE-Zertifizierung und jährliche Produktionskontrolle Modul D durch Certificación CE y control de producción anual Módulo D por Certificazione CE e controllo annuale della produzione Modulo D by

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EN Intended to protect against liquid or solid particles but not against gases and vapours. The masks are necessarily classified according to three categories FFP1 - FFP2 or FFP3. Our foldable masks are individually packaged and in a dispenser box. Our cup masks are packaged in a blister or a dispenser box.
N.B. Before use the respirator should be fitted to the wearer for suitability by a competent Fit Tester.

For masks - series A : check the presence of the nose bar, check that the nose bar does not pierce the filter media. Open the mask and check for the presence of two elasticated straps. Take a mask in one hand (the outer face of the mask in your palm). Preform the nose bar using your thumb and index finger. With the other hand pass the first strap to the back of the head to under the ears. Take the second strap and place it on the top of the head. Adjust the nose bar against the contours of the nose by molding it so that it fits perfectly and does not allow air to pass between the mask and the skin.

For masks - series A and B: Adjust the entire perimeter of the mask to each contour of your face to ensure the best possible seal. To test the tightness, exhale sharply. If the air protrudes at the level of the nose readjust the barrette as best as possible. If the air exceeds the contour of the mask, readjust the elastics towards the back of the head. The precaution taken when fitting the mask will determine its effectiveness and pay maximum attention to it. Do not hesitate to readjust the mask as necessary. If the mask still does not fit, do not enter an area where wearing a mask is required. These masks have been designed to allow glasses to be worn: put on the glasses after fitting the mask, at the level of the nasal bridge. The mask will only provide the protection claimed if the above recommendations are scrupulously respected.

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For masks - series B :

a/ Check the presence of two rubber straps. Take the mask in one hand, the outer face of the mask in the inside of the palm and the two rubber bands located on the outside of the hand.

b/ Position the mask on the face while inhaling, while ensuring that the nasal veil is well inside the mask.

c/ With the other hand, pass the upper elastic to the top of the head.

d/ Take the second rubber strap and pass it over the head to the nape of the neck. Adjust the bottom of the mask by pulling on it so that it covers the entire chin. For a comfortable fit, ensure that the elastics are not rolled up on themselves. Make sure that the nasal veil is turned inside the mask and that it fits in this direction, on the bridge of the nose. If it turned around during the application of the mask, make sure to put it back in place to adapt to the bridge of your nose. Perform the operation until you no longer feel air on your eyes when exhaling.

*EN - The air proof lining has to be placed properly to avoid any leakage. If you have any doubt about fitting, please feel free to contact the manufacturer for more explanations and demonstration.

NR : Not re-usable NR D : Not re-usable and Complies with clogging test with dolomite dust

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WARNING - To be read and understood before use.

HOW TO USE - Prior to use : verify that the mask is not damaged. If the mask is damaged and to ensure full traceability, contact the manufacturer with the contact details above, noting the batch number and the box number if possible for the largest packaging.

1. Never attempt to modify this mask which has been designed and certified for your safety.
2. Use the category of mask corresponding to your need and level of protection, depending on the environment in which you will work. Always refer to health & safety at work to define the appropriate protection required.
3. This mask is for single use only. After a maximum of 8 hours of continuous use, subject to point 7 below, dispose of it following the guidance corresponding to the type of harmful substance it has been in contact with. It should also be changed and discarded any time it has been removed from the face, or when you have difficulty breathing due to significant breathing resistance due to clogging.
4. Avoid excessive humidity. Shelf life: refer to the smallest packaging.
5. As this mask does not supply oxygen, use it in a place with sufficient ventilation. If the oxygen content of the air is less than 17%, immediately return to a well-ventilated area.
6. Never use this mask unless you know exactly the nature of the harmful environment you are likely to encounter. Do not use in explosive environment.
7. It is up to the user alone (and in no case to the manufacturer) to ensure the suitability of the mask with the characteristics of the environment of use. The recommended duration of use is maximum and must be adapted by the user according to the storage conditions as well as the characteristics of use (temperature, pressure, hygrometry, concentration of particles to be filtered, morphology of the user, efforts made ...).

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